



Check for signs of Sudden Cardiac Arrest: Unresponsive
 Not Breathing Normally


1 PULL

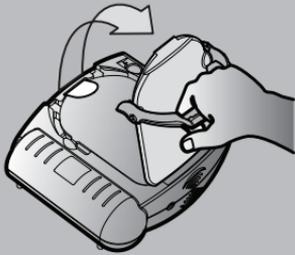

2 PLACE

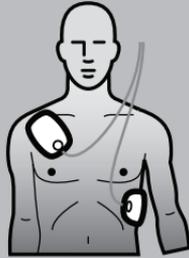



3 PRESS

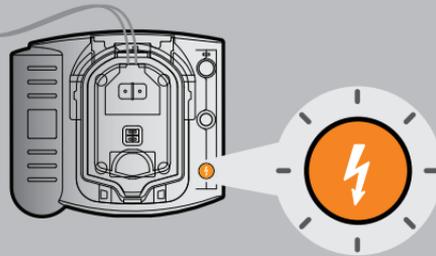

QUICK REFERENCE GUIDE


DETAILED INSTRUCTIONS

To start: Pull handle. Remove protective cover.

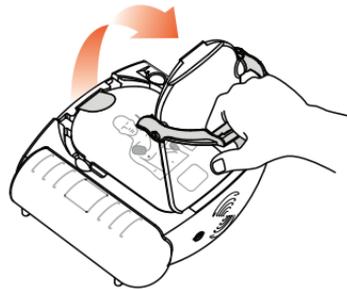
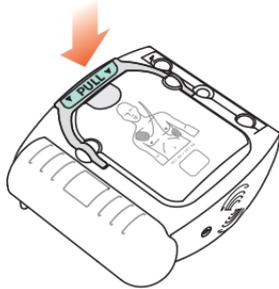


1

Remove all clothes from victim's chest.
Clean and dry skin, if needed.


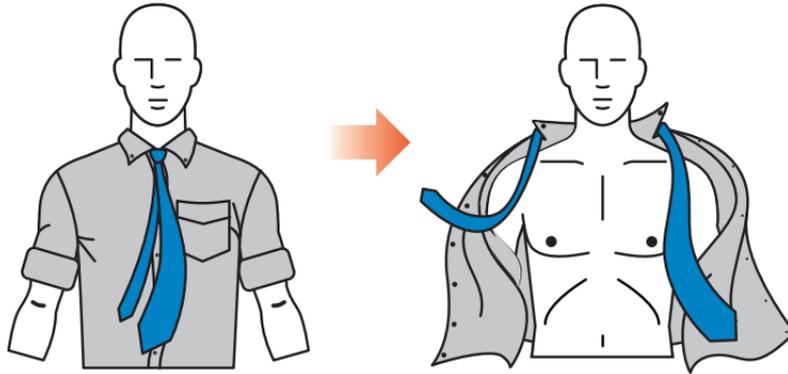


Peel off film seal and remove pads.


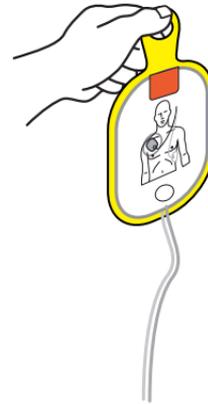
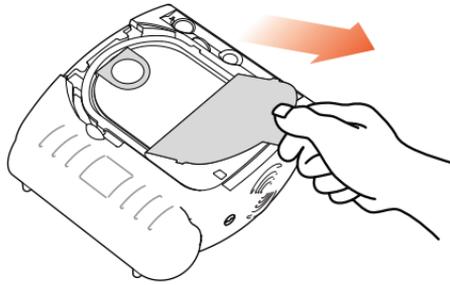


3

Peel each pad from yellow plastic liner.

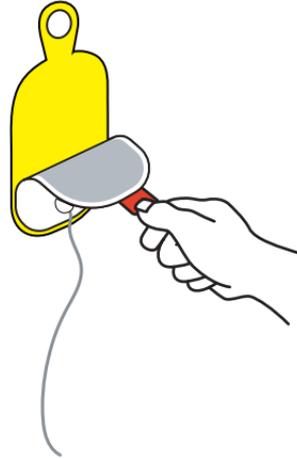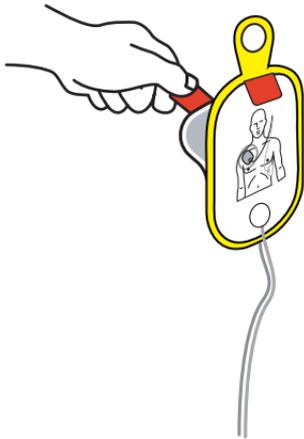


Place pads exactly as shown in the picture.

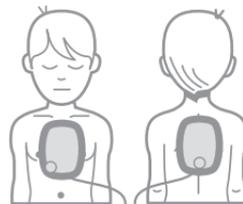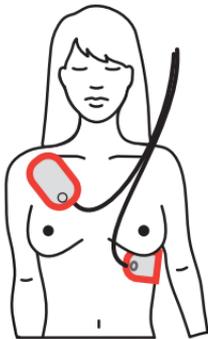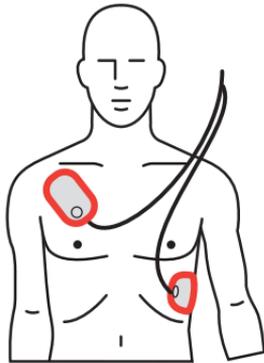


(0-8 YEARS) < 55 lbs/25 kg


Analyzing ... Do not touch the patient!

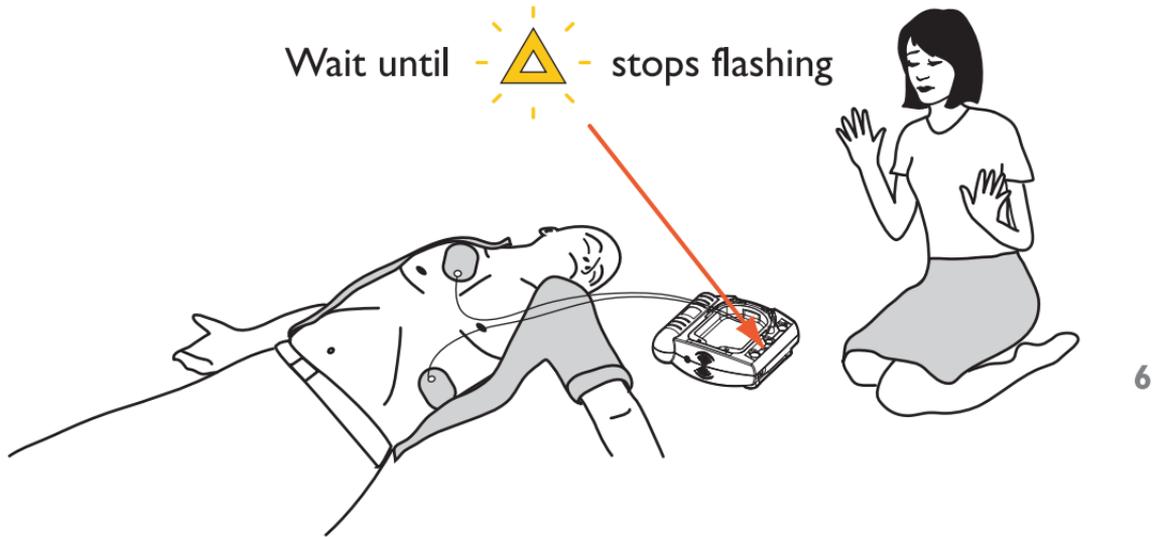

To shock: Press flashing **ORANGE** shock button.
Do not touch patient.

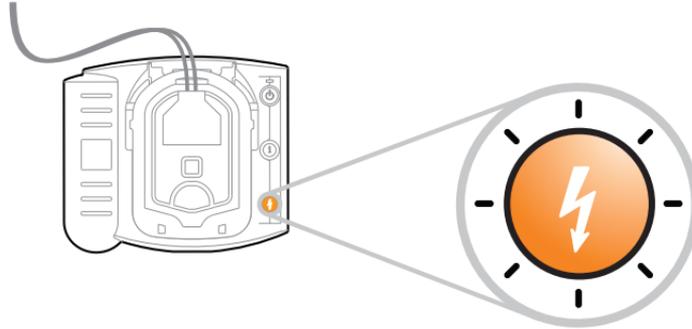


To get help with CPR: Press flashing **BLUE** button.

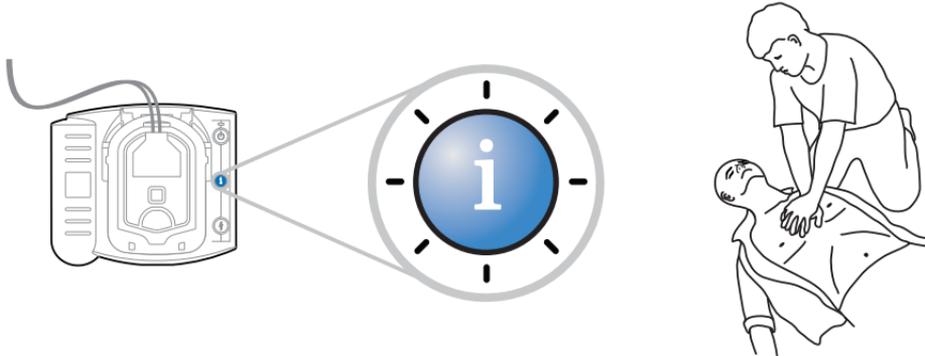


30 compressions • Pinch nose, tilt head • 2 full breaths

To turn ON: Press **GREEN** on/off button.

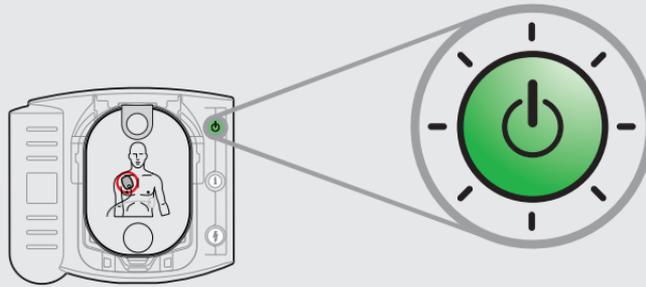

If cartridge needs to be replaced: Slide latch.

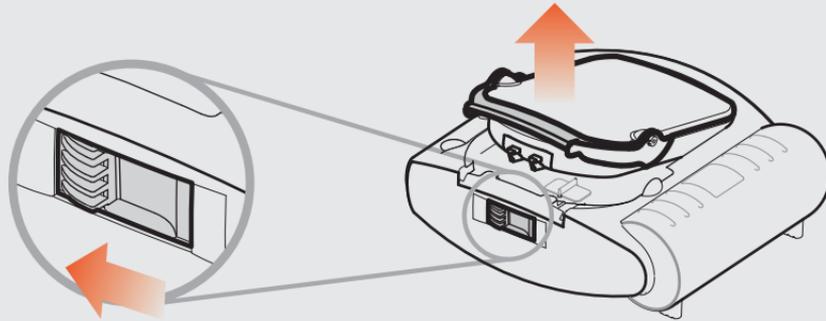

Printed in USA

Philips Medical Systems
Seattle, WA 98121

Copyright © 2008
Philips Electronics North American Corp

011834 REV XIV

M 5 0 6 6 - 9 7 8 0 0



0 1 1 8 3 4 - 0 0 0 8

PHILIPS